

WHAT ARE THE SPECIFICATIONS OF THE NEEDLE ON A DULAGLUTIDE PEN?

The dulaglutide prefilled pen is manufactured with a 29 G (0.33 mm) x ½" (12.7 mm) needle. All dulaglutide prefilled pens have the same needle size.¹

The needle is permanently attached to the pen and cannot be taken off.

- Therefore, it is not possible to attach spare needles or safety needles.¹

The injection depth of the needle is approximately 5 mm.¹

The needle is lubricated.¹

HOW CAN THE USER AVOID DAMAGING THE NEEDLE?

Make sure the pen is locked.

Pull off the grey base cap. Do not put the base cap back on – this could damage the needle.

- Discard the base-cap.

Do not touch the needle.²

REFERENCES

1. Data on file, Eli Lilly and Company and/or one of its subsidiaries.
2. Trulicity [instructions for use]. Eli Lilly Nederland B.V., The Netherlands.